



# Instruction Sheet for the Biomedical Polymers Disposable Cytology Funnel Clip

(patent pending)

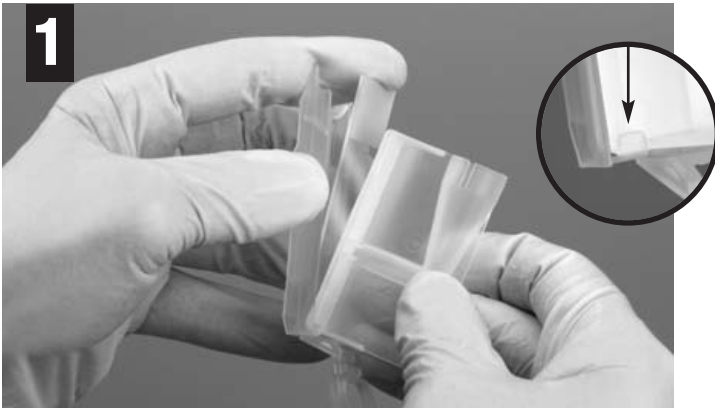


**1** With the funnel clip in the open position, place the microscope slide and then the cytology funnel into the clip. Be sure that the slide and funnel are behind the two protruding supports at the bottom of the base of the funnel clip.

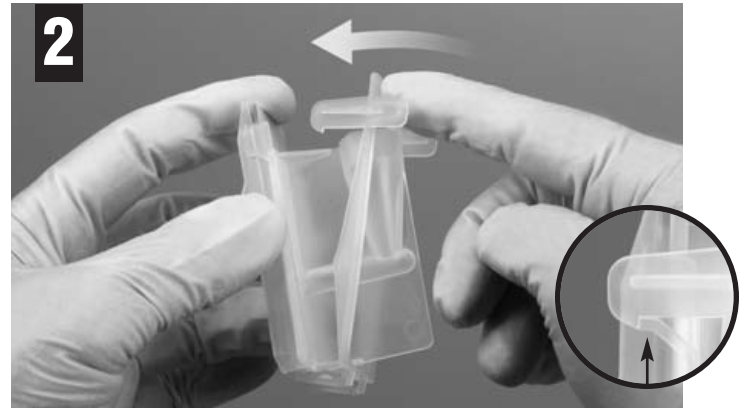

**2** Rotate the front section of the funnel clip up over the funnel and secure to the funnel base. Please ensure that both catches on the clip fully engage the catches on both sides of the funnel base.



**3** To remove the funnel after the centrifuge cycle hold the base by the sides with one finger on the top of the exposed microscope slide.

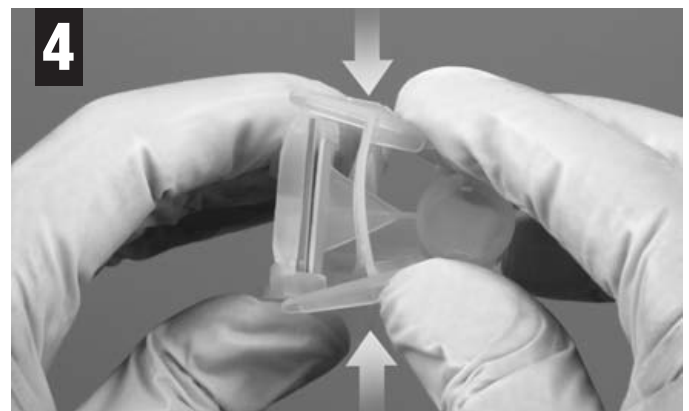

**4** Squeeze the catches of the front section of the clip to release it from the base. Swing open and remove slide for further processing and analysis. **Cytology funnel and clip assembly are disposable.**

Cat. No.	Description	Quantity
<b>BMP-CYTO-S50</b>	Single Cytology Funnel with White Filter Card	50/Pack
<b>BMP-CYTO-S500</b>	Single Cytology Funnel with White Filter Card (Bulk)	500/Case
<b>BMP-CYTO-S50BP</b>	Single Cytology Funnel with Brown Filter Card	50/Pack
<b>BMP-CYTO-DB25</b>	Double Cytology Funnel with White Filter Card	25/Pack
<b>BMP-CYTO-DB500</b>	Double Cytology Funnel with White Filter Card (Bulk)	500/Case
<b>BM-FUNNEL-CLIP</b>	Metal Slide Clips	6/Pack
<b>BMP-CYTO-PLCLIP</b>	Plastic Disposable Slide Clip	50/Pack

plastic products for laboratory & medical specimen handling



42 Linus Allain Avenue • Gardner, MA 01440 USA  
 Toll Free: 800/253-3684 • Tel: 978/632-2555 • Fax: 978/632-2524  
 E-mail: sales@biomedicalpolymers.com • www.biomedicalpolymers.com